

folded size: 103 x 60 mm FIR-B2 cover & backcover

Notice Before Use 注意事項

- This product is not suitable for use by pregnant women and babies.
- Children should not use this product without parental supervision.
- Do not heat or boil the product in a microwave oven.
- If allergic conditions or discomfort occur after use, reduce the time of use and consult your doctor immediately.
- 本產品不適用於孕婦及嬰兒。
- 兒童必須在家長監管下使用。
- 請勿將產品放進微波爐加熱或煮沸。
- 如使用後發覺有敏感或不適，請縮短使用時間，並立即諮詢您的醫生。

www.germanpool.com



**GERMAN
POOL**

USER GUIDE

使用指南

FIR-B2

FAR INFRARED NECK SUPPORT | 遠紅外線保健頸套

folded size: 103 x 60 mm FIR-B2 inside

Instructions For Use 使用方法

Wrap the Neck Support around your neck with the BIO-ACT Energy Layer (red in colour) facing inside. Adjust to prevent the product from becoming too tight to your neck and fasten the Velcro behind your neck. To ease cervical vertebrae pain, place the wider part at the back of your neck. To soothe itchy throat, place the wider part to the front.

紅色遠紅外線能量層向內，將頸套圍繞頸部一周，調整至理想鬆緊度後，貼上魔術貼即可。如針對頸椎痛問題，請將較寬的一面置於後頸；如針對喉嚨癢問題，請將較寬的一面置於前頸位置。



Care & Maintenance

1. Wash in warm water with a mild detergent. Gently squeeze product in water. Rinse thoroughly with clean warm water and drip dry.
2. Do not soak product in water for a long time.
3. If the product is too dirty, gently rub the soiled areas with a damp cloth. Repeat step 1.
4. Do not rub with force, especially in the FIR Energy Spot areas.
5. Frequent washing is recommended to keep product clean and free of body salts which may cause skin irritation.

清洗及保養方法

1. 用溫和洗衣粉於溫水中清洗，輕輕於水中按壓，再以清潔溫水沖淨後滴乾。
2. 切勿浸泡太久。
3. 如太骯髒，可先用濕布輕輕擦洗骯髒部份再重複步驟 1。
4. 切勿大力洗擦，尤其遠紅外線能量點 (FIR Energy Spot) 部分。
5. 建議經常清洗，以免身體鹽份或污垢積聚導致皮膚敏感。

